3, Petite rue
F- 55170 ANCERVILLE
Tél. : 03.29.75.20.80
Fax. : 03.29.75.21.56
<http://www.canule.com>
E.mail : jmasarl@wanadoo.fr
SIRET 378 293 757 00019 NAF 331B

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ENGLISH

You have just purchased an adapter for a multifunctional syringe, **JMA** thanks you for your purchase.

It's very important to read thoroughly this notice before using the JMA system.

INSTRUCTIONS FOR USE

JMA SYSTEM DESCRIPTION

JMA System is composed by an adapter and a cannula.

The **adapter** is adjusted by hand to the original multifunction syringe. There are several references for the adapters, depending on the type of syringe used by the practitioner. The adapter can be autoclaved.

The **cannula** is manually screwed to the extremity of the adapter.

NOT REUSABLE. The Cannula is a single use product, to be used only in one patient: the re-use can involve mechanic , physicochemical risks., and / or biological contamination.

The water flows from the adapter's tip and the air through the four holes in the rim: the air/water mixture takes place inside the cannula creating a powerful and homogeneous spray.

UTILISATION AND INDICATION

The **JMA System** is used in dental offices to pulverize blasts of air, water or a mixture of both into the mouth...

The main objective of this device is to prevent cross- contaminations in dental offices, by means of an autoclavable adapter and a single use cannula.

ADAPTER INSTALLATION

- Remove the tip of the original syringe.

- Replace it with the adapter.

CONTROL OF FUNCTIONALITY OF THE ADAPTER

- Check that the adapter is correctly positioned and firmly secured.

- The water should flow out of the end of the adapter and the air through the four holes in the rim. Should the opposite happen, please have your equipment checked..

Your multifunction syringe is now equipped to use **JMA** single use disposable cannulas.

CANNULA FITTING AND RECOMMENDATIONS

- **Screw the cannula by hand until you feel a slight resistance.**

- The cannula can only be screwed once into place. The cannula should not be reused as the fixation pins may get damaged as the cannula is retained by pins, these may get damaged so that they no longer secure the cannula properly. This can also lead the cannula to be ejected from its mounting during the use of the syringe.

PACKAGING, CLEANING, DISINFECTATION AND

STERILIZATION

JMA System is delivered non sterile.

To clean the **JMA** adapter, spray all surfaces of the device to remove any residues..

Disinfect with a clean damp cloth.

Products containing acetone, chlorine and bleach are not recommended as disinfectants.

Never immerse in disinfectant solutions.

Not suitable for ultrasonic bath.

Before sterilization, remove any residue and carefully clean the adapter.

The **JMA** adapter can be sterilized in an autoclave (by a validated method).

The cannula is for a single use and must be discarded in compliance with current laws and regulations.

CAUTIONS

Use in a different way as described in the instructions

A failure to adhere to these instructions discharges the manufacturer from any responsibilities.

Some technical issues you can meet:

-Lack of watertightness / airtightness and mixing of water/air at the level of four peripheral holes or air in the water at the end of the adapter : if necessary place new seals, if the problem persists, contact your seller or the **JMA** company.

-Should you experience problems in the coupling of the cannulas or joining rings, please return the corresponding parts to the seller (or directly to **JMA**) accompanied by a proof of purchase or original packing in order to proceed to have them repaired or exchanged.

-Problem of crimping assemblies: please return the corresponding parts to the seller (or directly to **JMA**) accompanied by a proof of purchase or original packing in order to proceed to have them repaired or exchanged.

Marking and guarantee

Each product has a batch number corresponding to its manufacturing process. This number enables to trace back all the manufacturing steps from the raw material to its final packing.

The adapter is guaranteed for one year from date of purchase. Should you need to return the product, please also its batch number.

Vigilance

Any adverse event must be reported to the **JMA** authorized representative.

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