3, Petite rue
F- 55170 ANCERVILLE
Tél. : 03.29.75.20.80
Fax. : 03.29.75.21.56
<http://www.canule.com>
E.mail : jmasarl@wanadoo.fr
SIRET 378 293 757 00019 NAF 331B

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ENGLISH
<p>You have just purchased an adapter for a multifunctional syringe. JMA thanks you for your purchase.</p> <p>It's very important to read thoroughly this notice before using the JMA system.</p>
INSTRUCTIONS FOR USE
<p><u>JMA SYSTEM DESCRIPTION</u></p> <p>JMA System is composed by an adapter and a cannula.</p> <p>The adapter is adjusted by hand to the original multifunction syringe. There are several references for the adapters, depending on the type of syringe used by the practitioner. The adapter can be autoclaved.</p> <p>The cannula is manually screwed to the extremity of the adapter.</p> <p>NOT REUSABLE. The Cannula is a single use product, to be used only in one patient: the re-use can involve mechanic , physicochemical risks,, and / or biological contamination.</p> <p>The water flows from the adapter's tip and the air through the four holes in the rim: the air/water mixture takes place inside the cannula creating a powerful and homogeneous spray.</p> <p><u>UTILISATION AND INDICATION</u></p> <p>The JMA System is used in dental offices to pulverize blasts of air, water or a mixture of both into the mouth...</p> <p>The main objective of this device is to prevent cross- contaminations in dental offices, by means of an autoclavable adapter and a single use cannula.</p> <p><u>ADAPTER INSTALLATION</u></p> <ul style="list-style-type: none"> - Remove the tip of the original syringe. - Replace it with the adapter. <p><u>CONTROL OF FUNCTIONALITY OF THE ADAPTER</u></p> <ul style="list-style-type: none"> - Check that the adapter is correctly positioned and firmly secured. - The water should flow out of the end of the adapter and the air through the four holes in the rim. Should the opposite happen, please have your equipment checked.. <p>Your multifunction syringe is now equipped to use JMA single use disposable cannulas.</p> <p><u>CANNULA FITTING AND RECOMMENDATIONS</u></p> <ul style="list-style-type: none"> - Screw the cannula by hand until you feel a slight resistance. - The cannula can only be screwed once into place. The cannula should not be reused as the fixation pins may get damaged as the cannula is retained by pins, these may get damaged so that they no longer secure the cannula properly. This can also lead the cannula to be ejected from its mounting during the use of the syringe. <p><u>PACKAGING, CLEANING, DISINFECTION AND STERILIZATION</u></p> <p>JMA System is delivered non sterile.</p> <p>To clean the JMA adapter, spray all surfaces of the device to remove any residues..</p> <p>Disinfect with a clean damp cloth.</p> <p>Products containing acetone, chlorine and bleach are not recommended as disinfectants.</p> <p>Never immerse in disinfectant solutions.</p> <p>Not suitable for ultrasonic bath.</p> <p>Before sterilization, remove any residue and carefully clean the adapter.</p> <p>The JMA adapter can be sterilized in an autoclave (by a validated method).</p> <p>The cannula is for a single use and must be discarded in compliance with current laws and regulations.</p>
CAUTIONS
<p>Use in a different way as described in the instructions</p> <p>A failure to adhere to these instructions discharges the manufacturer from any responsibilities.</p> <p>Some technical issues you can meet:</p> <ul style="list-style-type: none"> -Lack of watertightness / airtightness and mixing of water/air at the level of four peripheral holes or air in the water at the end of the adapter : if necessary place new seals, if the problem persists, contact your seller or the JMA company. -Should you experience problems in the coupling of the cannulas or joining rings, please return the corresponding parts to the seller (or directly to JMA) accompanied by a proof of purchase or original packing in order to proceed to have them repaired or exchanged. -Problem of crimping assemblies: please return the corresponding parts to the seller (or directly to JMA) accompanied by a proof of purchase or original packing in order to proceed to have them repaired or exchanged. <p>Marking and guarantee</p> <p>Each product has a batch number corresponding to it's manufacturing process. This number enables to trace back all the manufacturing steps from the raw material to it's final packing.</p> <p>The adapter is guaranteed for one year from date of purchase. Should you need to return the product, please also it's batch number.</p> <p>Vigilance</p> <p>Any adverse event must be reported to the JMA authorized representative.</p>

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